




RESPIRONICS

REMstar® Auto
M Series with A-Flex™

PROVIDER QUICK SETUP GUIDE

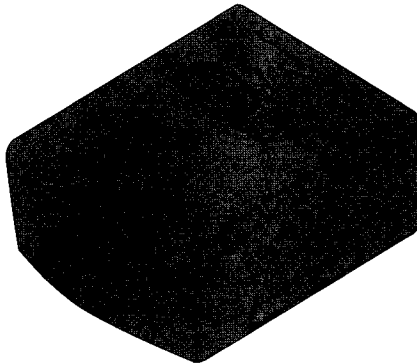



IMPORTANT! Remove this guide before giving the device to the patient. Only medical professionals should adjust pressure settings.

This guide provides you with basic instructions on how to navigate the provider setup and data menus to modify device settings. Refer to the device's provider manual for more information.

ACCESSING THE PROVIDER MODE SCREENS








Accessing provider mode unlocks additional settings that cannot be modified by the user. To access provider mode:

1. Plug the AC power cord into the power supply and an electrical outlet.
2. Hold down both the left and right   buttons while simultaneously plugging the power supply into the device. Once the device beeps twice, it is in provider mode and you can release the buttons. The Provider Mode Standby screen (shown on the next page) will appear.

You can view and modify prescription settings using the Provider Setup screens and view data about device usage on the Provider Data screens.

PROVIDER SETUP AND DATA MENU NAVIGATION

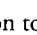To navigate the display screens from the Provider Mode Standby Screen:

- Press the  button to access the Provider Setup menu (Figure 1).
- Press the  or  buttons to access the Provider Data menu (Figure 2).
- Use the  button to navigate to the next screen and the  button to navigate to the previous screen.
- Use the  and  buttons to adjust the settings on the Setup screens.

Note: When you have finished modifying the settings, press the  button to exit Provider mode and return to User mode.

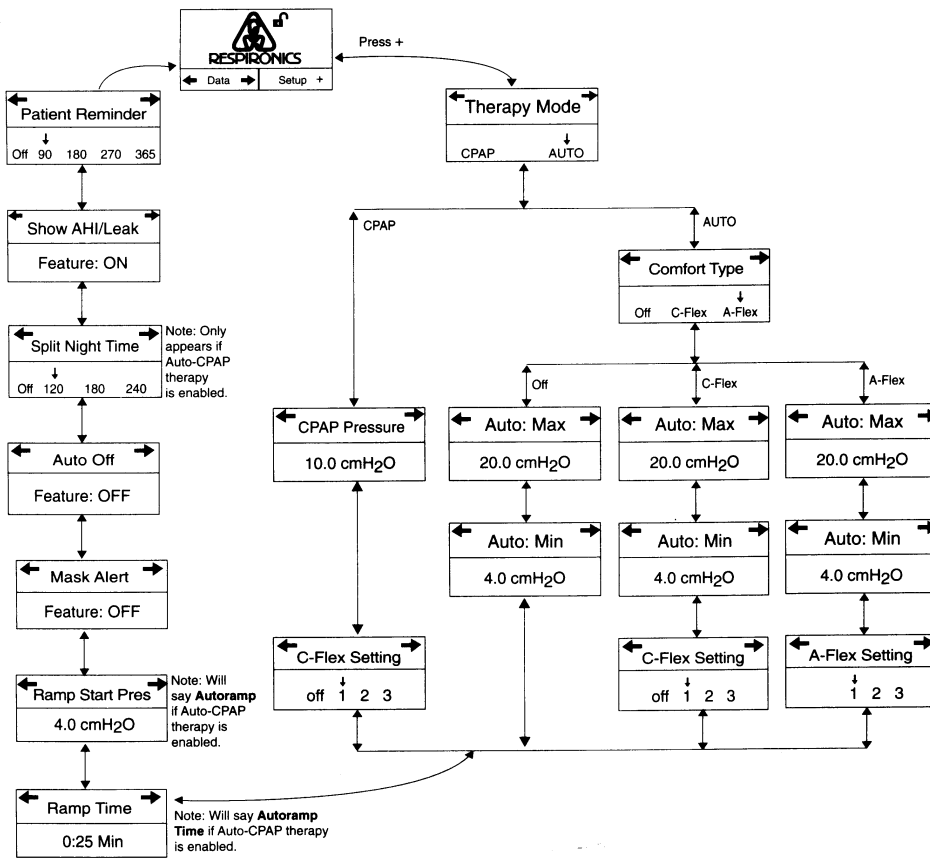


FIGURE 1 - PROVIDER SETUP SCREENS

ADDITIONAL SETTING INFORMATION

- **Mask Alert:** This alert will indicate when the device has detected a mask leak. You can enable or disable the alert.
- **Split Night Time:** This setting indicates the amount of time spent in CPAP therapy before transitioning to Auto-CPAP therapy (Off, 120, 180, or 240 minutes).
- **Show AHI/Leak:** You can select whether or not the Apnea/Hypopnea Index and System Leak averages display on the patient data screens.
- **Patient Reminder:** Using the Encore® Pro software, you can set a customized text message that will periodically display to the user. The Patient Reminder screen lets you specify how often the reminder will appear (every 90 days, 180 days, etc.).

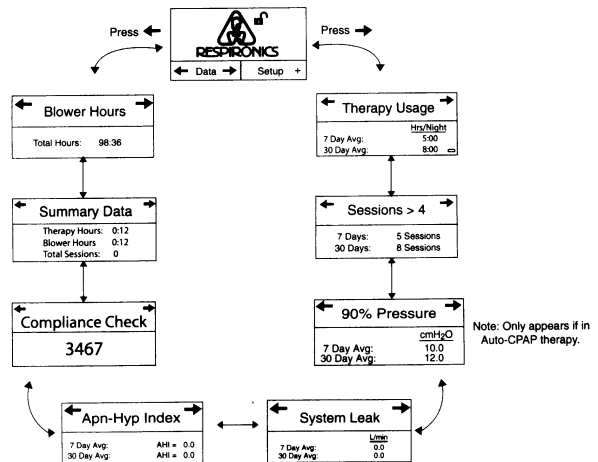


FIGURE 2 - PROVIDER DATA SCREENS



1001 Murry Ridge Lane
Murrysville, PA 15668
www.respironics.com

Respironics Deutschland
Gewerbestrasse 17
8221 Herrsching, Germany



1040515
JR 1/15/2007